

NIDEK 製品消毒

当社の製品の患者接触部（顎載せ台と額当て）、及び装置の外装はアルコールを含ませた布で拭き上げることは可能です。ただし、以下の行為は機器の故障の原因となりますのでご注意ください。

- ・カバーが歪むような強い力を加える
- ・同じ個所を何度も擦る
- ・アルコールが垂れるほど布にしみ込ませて拭く
- ・製品に直接アルコールをスプレーする

消毒用アルコールを消毒に用いる場合、濃度 70vol%以上で使用することが推奨されています。

（AAO の指針より）

※AAO のホームページ：<https://www.aao.org/headline/alert-important-coronavirus-context>

当社としては、装置のご使用の際には感染症に対して慎重に適用することを推奨しています。これは装置の使用を控えることも含みます。

Parts contacted with the patient (chinrest and forehead rest pad) and housing for NIDEK products can be wiped up with a cloth dampened with alcohol. However, please take care with caution that the following action will cause the device failure.

- applying a strong force to distort the housing
- rubbing the same part many times
- soaking the alcohol into the cloth so that the alcohol drips
- spraying alcohol directly on the products

When using the disinfectant alcohol solution for the purpose of disinfection, it is recommended to use the disinfectant alcohol solution with a concentration of at least 70vol% (the guideline of AAO: see *1)

*1 <https://www.aao.org/headline/alert-important-coronavirus-context>

We recommend using the device more carefully for infections. This includes refraining from using the device.